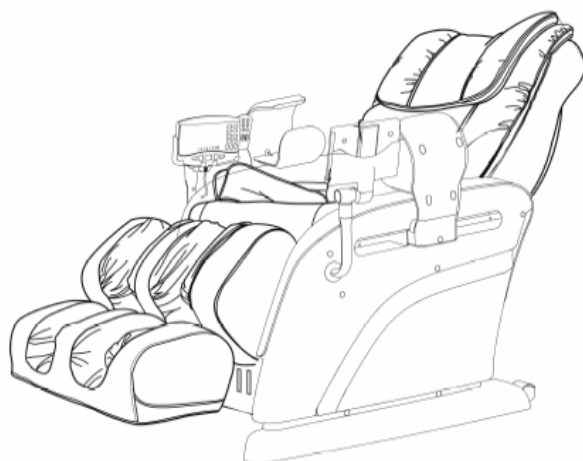
MOD. M-700

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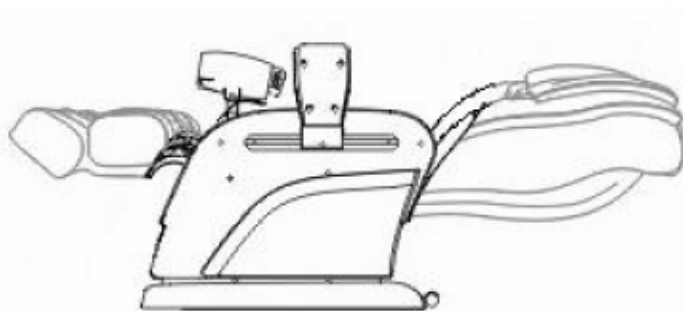
Massage Chair Ultimate Deluxe

CONTENTS

SAFETY PRECAUTIONS	1-2
Applicable surroundings	1
Unsuitable users	1
Instructions on using remote controller	1
Terms of using	2
FUNTIONS AND FEATURES	3
NAME AND FUNCTION FOR EACH PART	4-11
Exterior structure	4
Interior structure	5
Function areas on main remote controller	5
Instructions on buttons	6-8
VFD area descriptions	9-10
Instructions on auxiliary remote controller	11
PEPARATIONS BEFORE USE	12-13
Installation place	12
Floor protection	12
Surroundings	12
Grounding connection	12
Movement ways	12
Assembly method	13
OPERATING METHOD	14
Power on	14
Backward reclining	14
Forward reclining	14
MP3 music player	14
DVD/MPEG/TV player (optional)	15
AFTER USE	15
TROUBLE SHOOTING	16
CARE AND MAINTENANCE	17
SPECIFICATIONS	18

Massage Chair Ultimate Deluxe

● **SAFETY PRECAUTIONS**

- Before use, please carefully read “Important Safety Instructions” and use it properly.
- Please follow instructions carefully to avoid any possible personal injury or damage to the product.
- For safety reasons and in order to clearly indicate potential dangers, should the product be used improperly, instructions marked in the following manners must be strictly noted.

 WARNING	Improper operating may be seriously hurt to users.
 CAUTION:	Improper operating may be hurt to users or damaged to the product.

● **Applicable surrounding**

 CAUTIONS	To avoid any electrical shock or injury, do not use it in a bathroom or damp place!.
	To avoid fire, do not use product on or near any heating device or under direct sunshine!.
	To avoid fire, do not use product on electro-thermal carpet or other heaters!.
	Make sure to place the product on level and smooth surface at all time!.

● **Unsuitable users**

People suffering from the following conditions should consult with their doctor before use.

 PROHIBITIONS	People under treatment, injuries or abnormalities.
	Serious tumour patients.
	Pregnant women.
	Cardiac problems
	Osteoporosis patients
	Use of heart pacemakers or any other internal medical electrical apparatus.

● **Instructions on using remote controller**

 CAUTIONS	Do not operate the remote controller with wet hands.
	Do not throw onto the ground or under water.
	Do not pull connecting cord.
	Do not place anything heavy on remote controller.
	It is advisable to keep the remote controller on its original stand.

Massage Chair Ultimate Deluxe

- **SAFETY PRECAUTIONS**
 - **Terms of Using**

	When descending the footrest or backrest, use moderate pressure and be sure there are no obstructions and children near or under the footrest or backrest.
	Before use, uncover back cushion and make sure inside cushion is in good order together with all other parts. Should any part be damaged contact distributors immediately and do not use until repaired.
	To avoid any fire or damaged to the product, rated voltage must be used only.
	Keep supply cord unobstructed at all time, i.e. not twisted, knotted, over flexed or pulled and do not place anything heavy on it.
	If the supply cord is damaged, it must be replaced by the manufacturer, its service agent or similar qualified persons in order to avoid any hazard.
	Do not allow children or unconscious people to use product without any assistance.
	Do not allow children to play on the product.
	To avoid any injuries, do not use it immediately after meals.
	Do not use pointed or sharp objects on or near product at any time and avoid dragging on material of product.
	Make sure there is no any dangerous objects in pockets and arms i.e. cell phone, knife, watch, sharper to avoid any hurt.
	Do not use it while smoking to avoid fire.
	Do not use other medical apparatus when operating the product.
	Should any abnormalities occur when using, stop to consult with doctor.
	When disconnecting plug from power socket, be sure to hold insulating bush instead of supply cord.
	Move product on the fragile wood or carpet floor in case of any damage to the floor or truckle.
	Do not use it for medical purposes.
	After use, turn power off.
	Do not use petrol, caustic agents and spray pesticide for cleaning.
Do not sit on backrest, armrest and footrest to avoid any injury or damage.	
	Allow only authorized technicians to handle any breakdown, repair or alteration to the product.
	
	

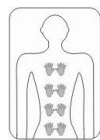
Massage Chair Ultimate Deluxe

• FUNCTION AND FEATURES


- 1.-The ergonomically shaped backrest moving structure design does more suit semasiology and offer best massage efficiency.
- 2.-Automatic body scanning technology effectively adjusts massage positions.
- 3.-Delicate massage presents humanistic and overall massage enjoyment. The backrest moving mechanism between neck and waist performs kneading, tapping, shiatsu and synchronization massage modes; Meanwhile adjustment of speed and width helps to relieve muscular fatigue and improve blood circulation.


Knead


Tapping

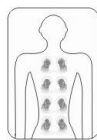
Synchronización

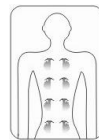Shiatsu

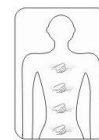Pummel

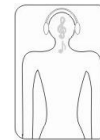
Rhythm

- 4.- Equipped with heating devices on waist, it effectively improves lumbar blood circulation.
- 5.-High power motors on perineum part offers strong vibration force.
- 6.-Feet bottoms are adopted wavelike massage and heating together to scatter cold and make you feel better.
- 7.-Lower back adopts wavelike air pressure massage.

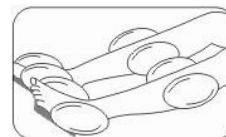Air bags on Both Sides of Thigh

Legs&Feet Air bags and Bottoms Heating

- 8.- Movable arms air pressure massage mechanism supplies alternative air pressure massage to arms.

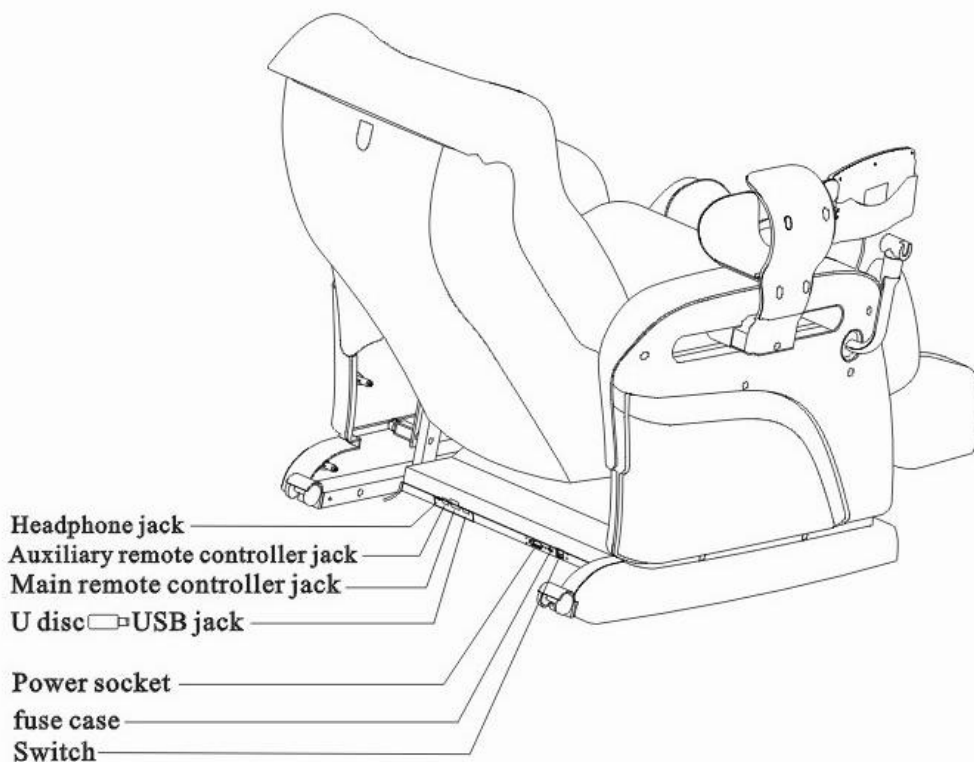
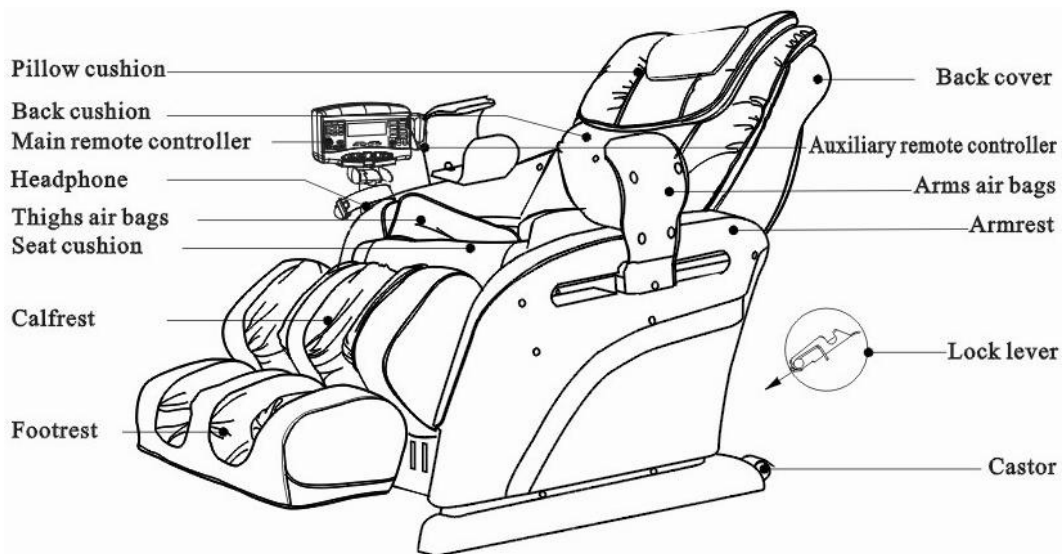
- 9.-Music interactive feature: music controls strength of backrest tapping and perineum vibration.

- 10.-MP3 music player introduces OTG technology which is very easy and convenient to renew songs.

Massage Chair Ultimate Deluxe


- NAME AND FUNCTION FOR EACH PART
- Exterior structure


Massage Chair Ultimate Deluxe

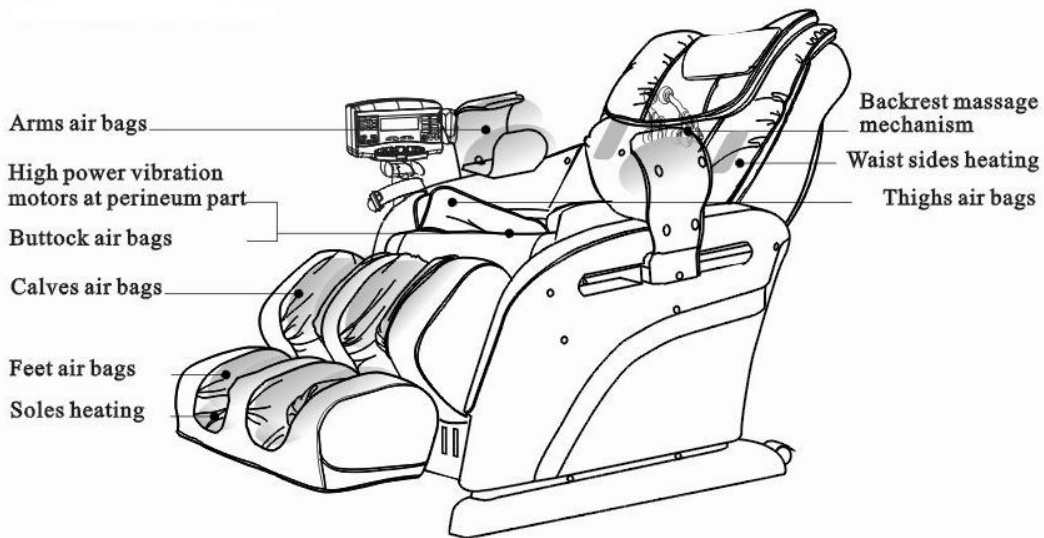- NAME AND FUNCTION FOR EACH PART
- Interior structure

- Function areas on main remote control

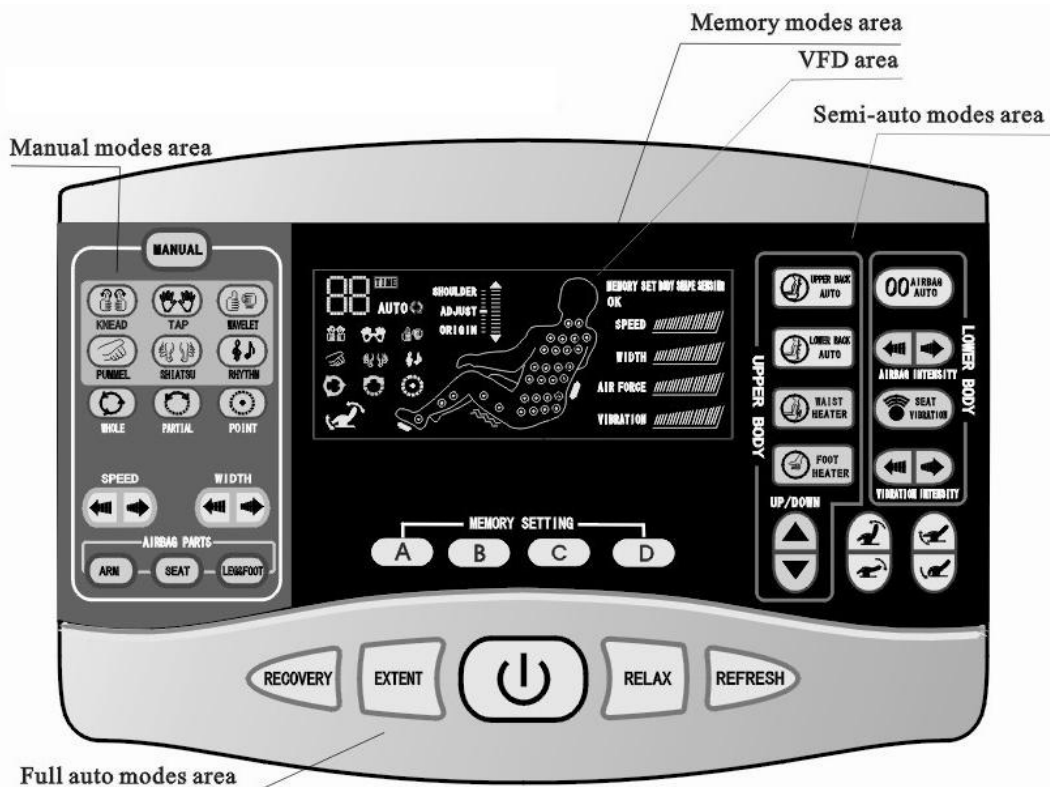Massage Chair Ultimate Deluxe

- NAME AND FUNCTION FOR EACH PART
- Instructions on buttons

Full auto modes area

Turn on/off massage functions.

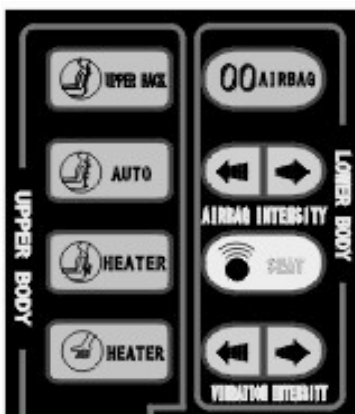
When power on. VFD is lightened on, function buttons and memory buttons are flashing for users selection.

When power off, massage chair resets, at this moment, all buttons are invalid. button keeps flashing to strat standby mode . After reset, VFD will be shut off.

Full automation:

There are four automation from RECOVERY, EXTENT, RELAX and REFRESH to cater for different ages of people.

❖ Semi-auto modes area

Neck & Shoulder AUTO: Select neck and shoulder automatic program.

Back & Waist AUTO: Select back and waist automatic programs.

Air bags AUTO: Turn on/off air bags automatic programs.

Air bags strength: There are five air forces for selection. When it reaches minimum/maximum point, a double sound alarm will be heard.

Turn on/off seat vibration.

Vibration intensity adjustment: There are five vibration intensities for selection. When it reaches minimum/maximum point, a double sound alarm will be heard.

Turn on/off back heating function.

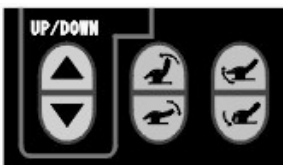
Turn on/off soles heating function.

Massage Chair Ultimate Deluxe

• NAME AND FUNCTION FOR EACH PART

• Ascend and Descend Adjustment

Back massage mechanism upwards button: valid only under shoulder adjustment and manual modes.
Back massage mechanism downwards button: valid only under shoulder adjustment and manual modes.

Backrest ascend button.
Backrest descend button.

Footrest ascend button.
Footrest descend button.

❖ Memory modes area

Four memory buttons from A,B,C and D can be selected repeatedly.

Memory setting: Under normal working status, keep pressing any button from A/B/C/D for 4 seconds at least, current function combination was stored. In the mean time, LED keeps flashing while VFD shows “memory complete” which means memory setting completes successfully.

Memory function starts: Press button, A/B/C/D four memory buttons are flashing, select any one of them, it will start the stored functions.

Note: body figure and shoulder inspection data can not be stored.

❖ Manual modes area

Massage Chair Ultimate Deluxe

❖ NAME AND FUNCTION FOR EACH PART

Manual modes buttons:

After selecting this button, the button is lightened and other buttons inside manual modes area are activated at the meantime.

• **Backrest massage modes selection**

Turn on/off kneading mode.

Turn on/off tapping mode.

Turn on/off kneading and tapping mode.

Turn on/off pummel mode.

Turn on/off shiatsu mode.

Turn on/off rhythm mode.

Back tapping will be synchronized with high frequency signal, while seat vibration.

❖ **Back massage locate**

Select overall massage position.

Select partial massage position.

Select point massage position.

❖ **Back massage speed adjustment**

There are five speeds for selection. Invalid for “shiatsu” massage mode. When it reaches minimum/maximum point, a double sound will be heard.

❖ **Back massage width adjustment**

There are three sections for selection. Invalid for “shiatsu” and “synchronization” massage modes. When it reaches minimum/maximum point, a double sound will be heard.

❖ **Air bags positions selection**

Turn on/off arms air bags massage.

Turn on/off seat air bags massage.

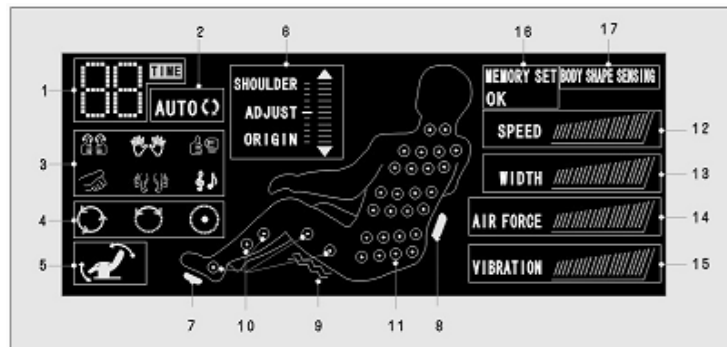
Turn on/off leg and foot air bags massage.

Massage Chair Ultimate Deluxe

❖ NAME AND FUNCTION FOR EACH PART

❖ VFD area descriptions

- 1. Time display: during working, it shows surplus time, the default time is 20 minutes
- 2. Backrest auto modes indications: it will be lightened under six backrest auto modes From "RECOVERY", "EXTENT", "RELAX", "REFRESH", "UPPER BACK AUTO"
- 3. Back massage modes: shows current back massage modes.

Knead

Tapping

Wavelet

Pummel

Shiatsu

Rhythm

●4. Back massage location mode

Overall mode: for the whole backrest.

Partial mode: on the basis of current position, massage area extends one third of overall.

Point mode: Pause at current point, desired massage points will be fulfilled by selecting "back upward and downward" button.

●5. Backrest and footrest ascend and descend

Footrest ascend and descend indication

Backrest ascend and descend indication

●6. Shoulder position adjustment and back massage mechanism resets

6.1. Back massage mechanism resets indicator.

6.2. Reference position: every interval distance keeps 2cm.

6.3. The massage mechanism position indicator refer to the reference position.

Massage Chair Ultimate Deluxe

• VFD area descriptions

- 7. Soles heating indication.
- 8. Waist sides heating indication.
- 9. Seat vibration indication.
- 10. Lower back air bags indication.
- 11. Upper back massage points and mechanism position indication.
- 12. Back mechanism massage speed indication: there are five speeds in total.

- 13. Back mechanism massage width induction: there are three widths.

- 14. Air strength indication: there are five air strengths.

- 15. Vibration intensity indication: five intensities.

- 16. Memory functions indication.

Memory functions on

- 17. Body figure scanning indication.

Massage Chair Ultimate Deluxe

❖ NAME AND FUNCTION FOR EACH PART

- Instructions on auxiliary remote controller

• MP3 music player instructions

Buttons

Play/pause current music.

Stop music and back to first song when replaying

Previous song.

Next song.

Turn up volume.

Turn down volume.

NOTE 1: The volume can change the effect of the music rhythm massage mode.

NOTE 2: Should any abnormality occurs on MP3 system or facilities have not been installed, a double sound will be heard when pressing the buttons.

Backrest and footrest ascend and descend buttons on auxiliary remote control fulfil the same functions as on the main remote control.

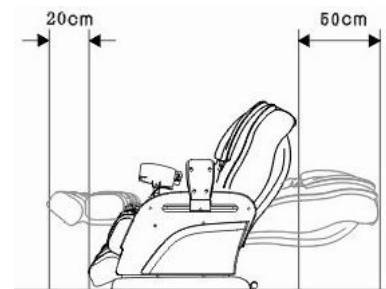
Massage Chair Ultimate Deluxe

❖ PREPARATIONS BEFORE USE

- **Installation place**
- **Safety distance around massage chair.**


Horizontal interval of backrest between up most and lowest point should be more than 50cm.

Horizontal interval of footrest between up most and lowest point should be more than 20cm.

Floor protection

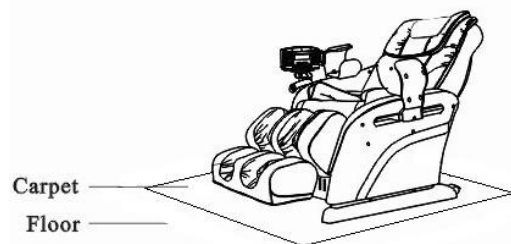The product could damage the floor, So a medium material under the chair (carpet for instance) is strongly advised.
To use for floor protection.

❖ Surroundings

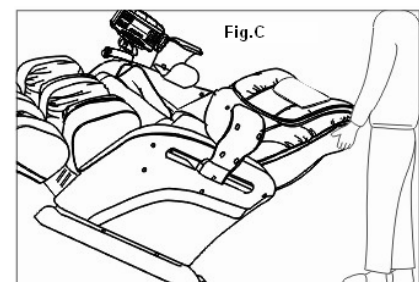
- ⚡ Do not keep the production in a bathroom or damp place to avoid electricity leaking.
- ☀ Do not keep the product near stove or heating devices to avoid fire or damage to the leather cover.

⚡ Grounding Connection

The product belongs to class I electric appliance. Please make sure to confirm ground Connection to avoid electricity leaking or any accident when using.

❖ Movement ways

Lift the front of the chair by using footrest and push with two hands to start moving Fig.C. When moving chair do not have persons or objects on chair to avoid overturning; Do not move or drag the massage chair on a delicate floor; Do not move chair by holding on to the seat or backrest, footrest must be used only; The chair is heavy, so handle with care.

Massage Chair Ultimate Deluxe

❖ PREPARATIONS BEFORE USE

• Assembly method

Step 1:
Unpack the carton
and take chair out
of carton by at least
four people

Prohibit holding here

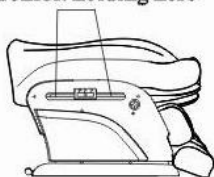
Fig 1

Step 2:
Unfolding the backrest to 115°,
a reminding sound means backrest locked.

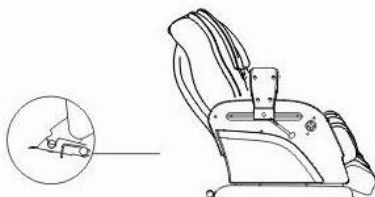
Fig 2

Step 3:
Connect air bag tubes and stick seat cushion

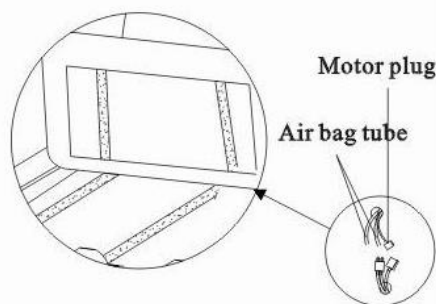
Fig 3

Step 4:
Fix the pillow cushion with
backrest by using of zipper

Zipper

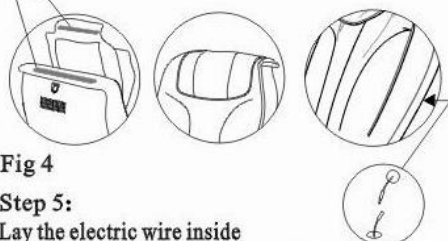
Fig 4

Step 5:
Lay the electric wire inside
the remote contro`
and fix the remote
bracket with four
screws.

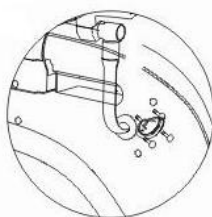
Fig 5

Step 6:
All above steps are done, connect main remote
control, auxiliary remote controller, DVD/MPEG
/ TV player and U disc into the related jacks, then
connect power, switch on and start working.

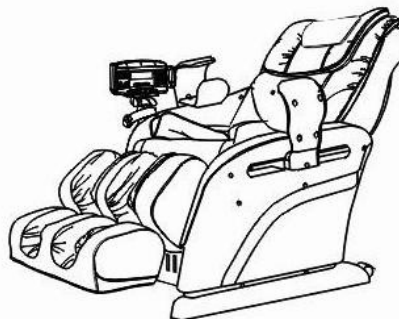
Fig 6

Massage Chair Ultimate Deluxe

❖ OPERATING METHOD

❖ Power on

- Connect plug to power socket.
- Switch on.

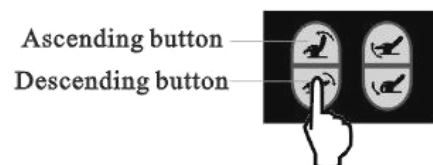
❖ Backward reclining

Keep pressing  button, backrest gradually goes down while footrest comes up with double sound reminding. When it reaches desired angle, release  button, backrest and footrest stop moving.

When backrest and footrest reach up most and lowest position separately, a reminding sound occurs. If backrest reaches up most, while footrest is still ascending, a continuous sound will occur, please keep pressing the button, footrest will ascend till up most point. Meanwhile footrest can be adjusted separately by  button and  button. During footrest ascend/descend, backrest keeps still, when footrest reaches up most/lowest point, a continuous sound will be heard.


Backward reclining

❖ Forward reclining

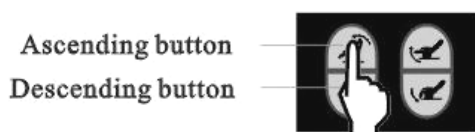The operating method is the same as “backward reclining”

Forward reclining

Massage Chair Ultimate Deluxe

❖ OPERATING METHOD

❖ MP3 music player

- The product include MP3 music player inside, either speaker or headphone are available to enjoy music.
- Connect U disc with songs into USB jack.
- 5-10 seconds later, please operate auxiliary remote control to play music.

❖ After use...

- After use, turn off power and remove plug from power socket.
- Keep the remote controller in the bracket.
- To avoid any accidents or damage, make sure to turn off power and remove plug from power socket.
- Should the product not be used for a long time, store the product.

Storing way:

- Firstly reposit footrest and backrest to the lowest and up most position separately, turnoff all massage functions and remove plug from power socket, then release lock lever as figured to lay the backrest onto the seat cushion.

Massage Chair Ultimate Deluxe

❖ TROUBLE SHOOTING

Symptoms	Please check below:	Solutions
“xiu xiu” or “da da” sound	It is attributed to the construction of the chair or from friction between the massage rollers and the leather cover.	Normal
Not operating	If plug has slipped out of power socket?.	Connect plug .
	If power has been switched off .	Switch on .
	If fuse was broken .	Replace same type of fuse .
Abnormal playing for certain music .	Does music format match MP3	Replace music format .
Double sound be heard when pressing buttons	Should U disc be connected well?	Reconnect U disc
	U disc has been initializing?	Select buttons after 5-10 seconds
	Is there any MP3 format music inside U disc?	Check it by computer
Soundless of speaker	If headphone still stays headphone connector	Pull out headphone
	If it is the lowest volume or turns off volume	Turn up volume
Over-current on motors.	Is it over-burden on backrest and calf shelf?.	Decrease burden on backrest and calf shelf

If above solutions have not solved the problem, turn off power, remove plug from power socket and contact the local distributors or headquarter for repair and maintenance.

Massage Chair Ultimate Deluxe

❖ CARD AND MAINTENANCE

Always remove plug from power socket when performing maintenance on chair.
Do not remove plug from power socket with damp cloth or wet hands to avoid accident.

❖ Maintenance method

Main body

To move dirt from plastic parts and armrests, Wipe with damp cloth using mild cleaning agent And dry thoroughly with clean cloth.

NOTICE

Do not use alcohol, caustic agents, detergents etc. to avoid damage, discolouring, splitting etc.

Remote controller

Wipe remote controller with dry cloth only.

NOTICE

Never use damp materials to clean remote controller to avoid accidents.

Back Cushion and Pillow

If dirty, clean with damp cloth and mild cleaning agent, followed with a clean damp cloth and leave to dry.

Storing Method

After dusting and cleaning, keep chair in a dry place. If stored for long time, cover to avoid spoilage.

NOTICE

Avoid leaving in direct sunlight, or in excessive heat as this will cause fading and colour change

Inspection

 Please inspect	<p>Following Phenomena?</p> <ul style="list-style-type: none"> Parched smell Periodically Insensitive Touch Electric wire heating Other abnormal phenomena 		<p>In case of accident, please pull off the plug and contact agent or our company. Do not allowed to disassemble or fix by yourself.</p>
---	--	--	--

Massage Chair Ultimate Deluxe

❖ SPECIFICATION

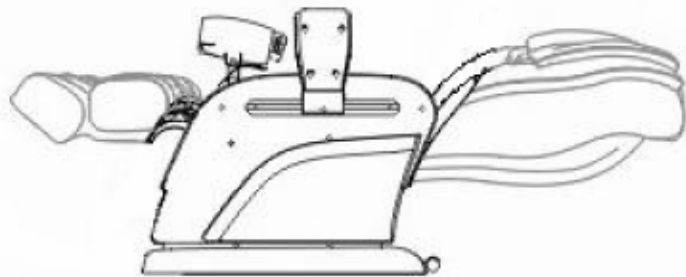
• **Description: Deluxe Multi-functional Massage Chair**

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